



NDI Passive Spheres™



EN

For use as an accessory to aid in the auto-registration and localization during computer-assisted therapy.

Do not re-sterilize. Re-sterilization may not be effective and may affect accuracy.

Do not use if sterile packaging is damaged.

1. Place sphere on mounting post and press down until snap occurs. (A)
2. If reflective surface becomes damaged or occluded with foreign matter, replace sphere before use.
3. Do not replace sphere while tool is in patient's body.
4. To remove sphere, grasp near base and pull upwards. (B)

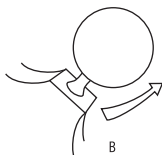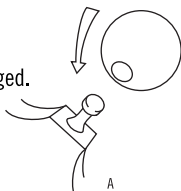

CAUTION: USA Federal law restricts this device to sale by or on order of a physician.



Northern Digital Inc.
103 Randall Drive
Waterloo, ON, Canada
N2V 1C5

STERILE EO



CE
2797

CE-Kennzeichnung gemäß Richtlinie 93/42/EWG
CE-marked according to directive 93/42/EEC
Marquage CE conforme à la directive 93/42/CEE
Identificación CE en conformidad con la directriz 93/42/CEE
Marchio CE conforme alla direttiva 93/42/CEE

EC REP NDI Europe GmbH

Güttinger Str. 37, 78315 Radolfzell, Germany
Toll Free + 800 634 634 00