Disposable Reflective Marker Spheres 41773/41813

ENGLISH

User Instructions:

For use as an accessory to aid in the auto-registration and localization during computer-assisted therapy.

Warning:

Ensure that the marker spheres can be screwed onto the pin until there is no gap between the marker sphere and the base of the pin. Do not use a marker sphere if it is not securely attached in this position. If excessive force has to be applied in order to screw a marker sphere onto the pin, the marker sphere should be disposed of, and a new marker sphere used instead.

Only use clean and dry reflective marker spheres. If the surface of the reflective marker sphere is soiled, wet, or contaminated with blood, this will result in inaccurate navigation.

To ensure accurate navigation, wet or soiled marker spheres must either be cleaned and dried before further use, or replaced with new marker spheres.

Only use a soft cloth moistened with sterile water to clean the surface of soiled marker spheres. Ensure that the cleaned marker sphere is absolutely dry before use.

If a marker sphere has been cleaned or replaced, the accuracy of the navigation system must be verified before navigation can continue.

If you use marker spheres, in the vicinity of oscillating or vibrating instruments, you must check the marker spheres at regular intervals to make sure that they are securely attached.

Caution:

Do not use defective or deformed marker spheres.

Disposable reflective marker spheres are highly sensitive medical devices. They should be handled with care and not dropped.

For further information on the use of the NDI Disposable Reflective Marker Spheres with a Brainlab IGS navigation system, please see the Brainlab user manual or consult Brainlab customer support.

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